50 YEARS
YASARGIL CLIPS

NEUROSURGERY

CLIP TRAY

FOR AESCULAP® YASARGIL ANEURYSM CLIPS
FÜR AESCULAP® YASARGIL ANEURYSMEN-CLIPS

CLIP TRAY

FOR AESCULAP® YASARGIL ANEURYSM CLIPS

VALIDATED AND EASY HANDLING

INDIVIDUAL CLIP-ARRANGEMENT INDIVIDUELLE CLIP-ANORDNUNG

Temporary and permanent clips can be stored in one tray for a rapid retrieval of suitable clips during surgery. The tags can be arranged individually corresponding to your personal preferences.

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The tray is made of metal which is more durable than current plastic trays even after hundreds of reprocessing cycles.

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An increased number of compartments in smaller trays saves space in the OR. The tray is available in two sizes. The small tray offers 21 compartments, the large tray has 42 compartments.

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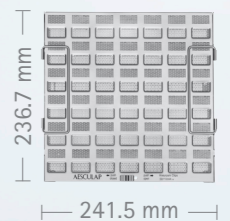
A Trace Plate makes the tracking of each clip easier. This Trace Plate is used to store the patient labels of the permanent clips that are in the clip tray.

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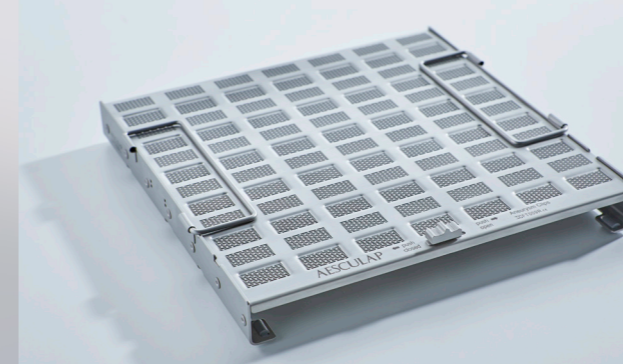
The tray provides a validated, clear and easy way to organize, store and **reprocess** your AESCULAP® YASARGIL Aneurysm clips.

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
VALIDATED REPROCESSABILITY OF ANEURYSM CLIPS INSIDE THE TRAY

- Validated reprocessing process: Cleaning and sterilization of temporary and permanent aneurysm clips inside the tray is validated. Tests show good results with regard to cleaning, biocompatibility and endotoxines.
- Good drying properties: The tray is made of stainless steel that prevents the appearance of residual water.
- Improved perforation design: The special design of the perforations of the tray offers a good water permeability.
- Long-term stability: Usage of high-alkaline cleanser is possible.

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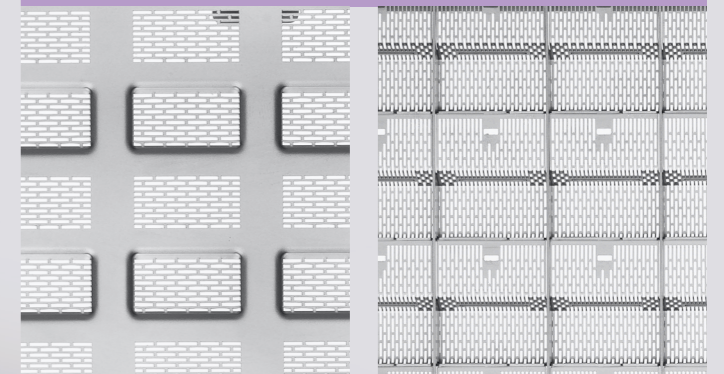FURTHER INFORMATION

LONGER COMPARTMENTS FOR LONG CLIPS LÄNGERE MULDEN FÜR LONG CLIPS

The AESCULAP® „long clips“ require longer compartments. This is being achieved by removing a separator and replacing it by a shorter separator.

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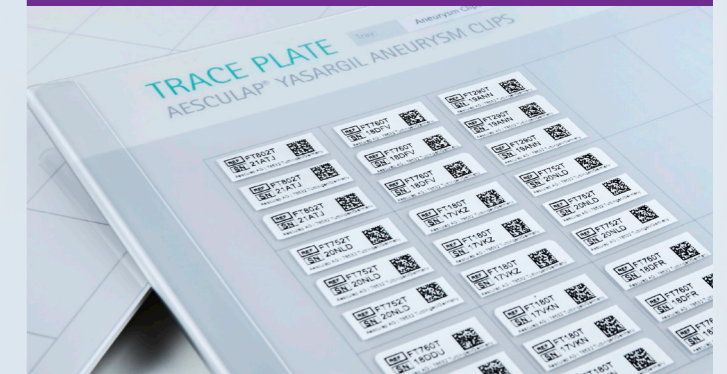
For your individual clip arrangement, a corresponding clip tag is available for each clip. The tag can be attached to the tray easily. The identification tags contain the article number, the blade length, the spring size as well as a picture in original size of each clip.

For an easy allocation of the clips to the tags, the identification tags have the same color coding as the AESCULAP® YASARGIL Aneurysm clips. Tags for temporary clips are yellow, tags for permanent clips are grey.

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TRACE PLATE FOR PATIENT LABELS TRACE PLATE FÜR PATIENTENLABELS

A Trace Plate for the storage of the patient labels is available. It represents the design of the small / large tray.

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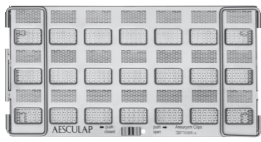
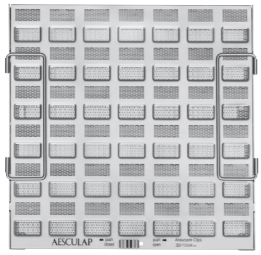
The identification labels make an individual marking of the trays possible.

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The main product trademark "Aesculap" is a registered trademark of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

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B. Braun Medical N.V./S.A. | Lambroekstraat 5b | 1831 Diegem | Belgium