# AESCLAP® ELAN 4

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# AESCLAP® ELAN 4

## APPLIED PARTS – INTERNAL CLEANING USING A RINSING DEVICE

Never use the rinsing device without a GB693 water filter.

**1.** Mount the applied parts (with tips pointing downwards) onto the rinsing device

**2.** Pre-cleaning:

Apply water gun to the side connector and rinse the applied parts at least 3 times for 5 seconds.

At the same time, use a compress to cover the connection for the irrigation tube.

**3.** Connect the rinsing device to the rinsing bracket in the washer-disinfector

**4.** Clean within washer-disinfector

**5.** Apply air gun to the side connector and blow the remaining water out of the applied parts

At the same time, use a compress to cover the connection for the irrigation tube.

**6.** Oil the applied parts

Remove applied parts from rinsing device and spray through with Sterilit® Power Systems for 2 seconds.

**7.** Blow impurities out of the water filter via the side connector

At the same time, use a compress to cover the connection for the irrigation tube.

The operator bears sole responsibility for the success of sterile reprocessing. Please pay particular attention to the directions and provisions in the instructions for use as well as the relevant national legal requirements and standards.