

Monosyn[®]

YOUR CHOICE FOR SOFT TISSUE APPROXIMATION

Mid-term synthetic absorbable monofilament suture
made of glyconate

Monosyn®

YOUR CHOICE FOR SOFT TISSUE APPROXIMATION



A COMPLETE RANGE OF SUTURES

B. Braun offers a comprehensive synthetic absorbable monofilament portfolio that provides solutions according to surgeon's requirements for the different surgical layers:

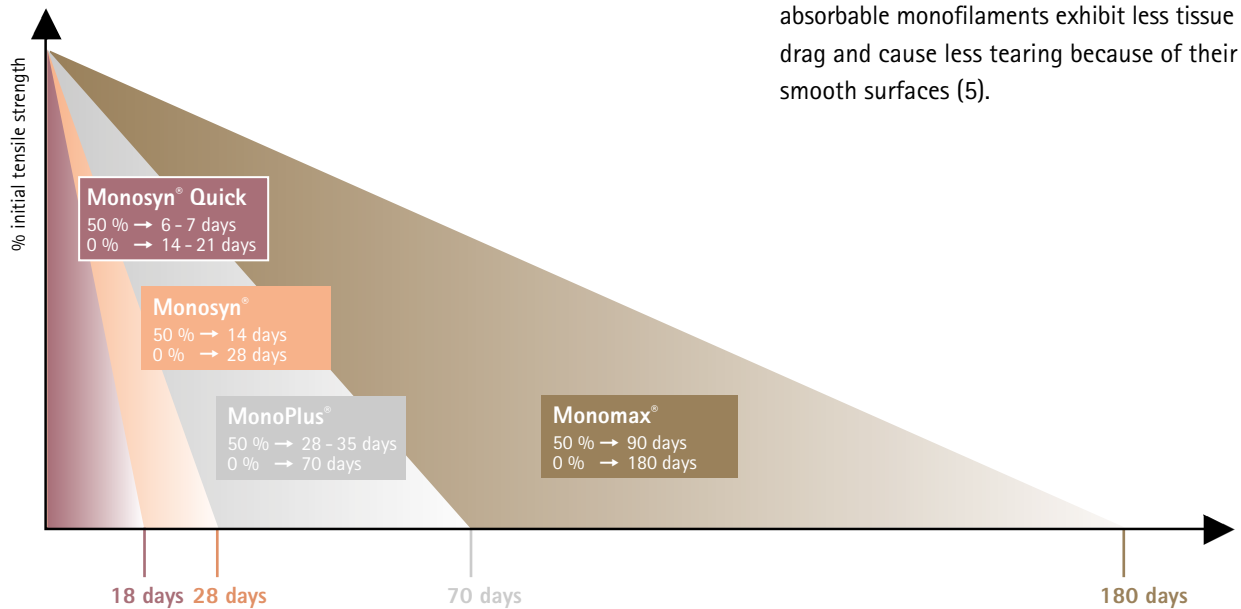


Figure 1: Tensile strength retention of the B. Braun synthetic monofilament sutures

Monosyn® Quick: Short term absorbable suture for fast healing tissues.

Monosyn®: Mid term absorbable suture for soft tissue approximation.

MonoPlus®: Long term absorbable suture designed for slow healing tissues.

Monomax®: Extra long term absorbable suture specially engineered for abdominal wall closure.

THE MONOFILAMENT ADVANTAGE

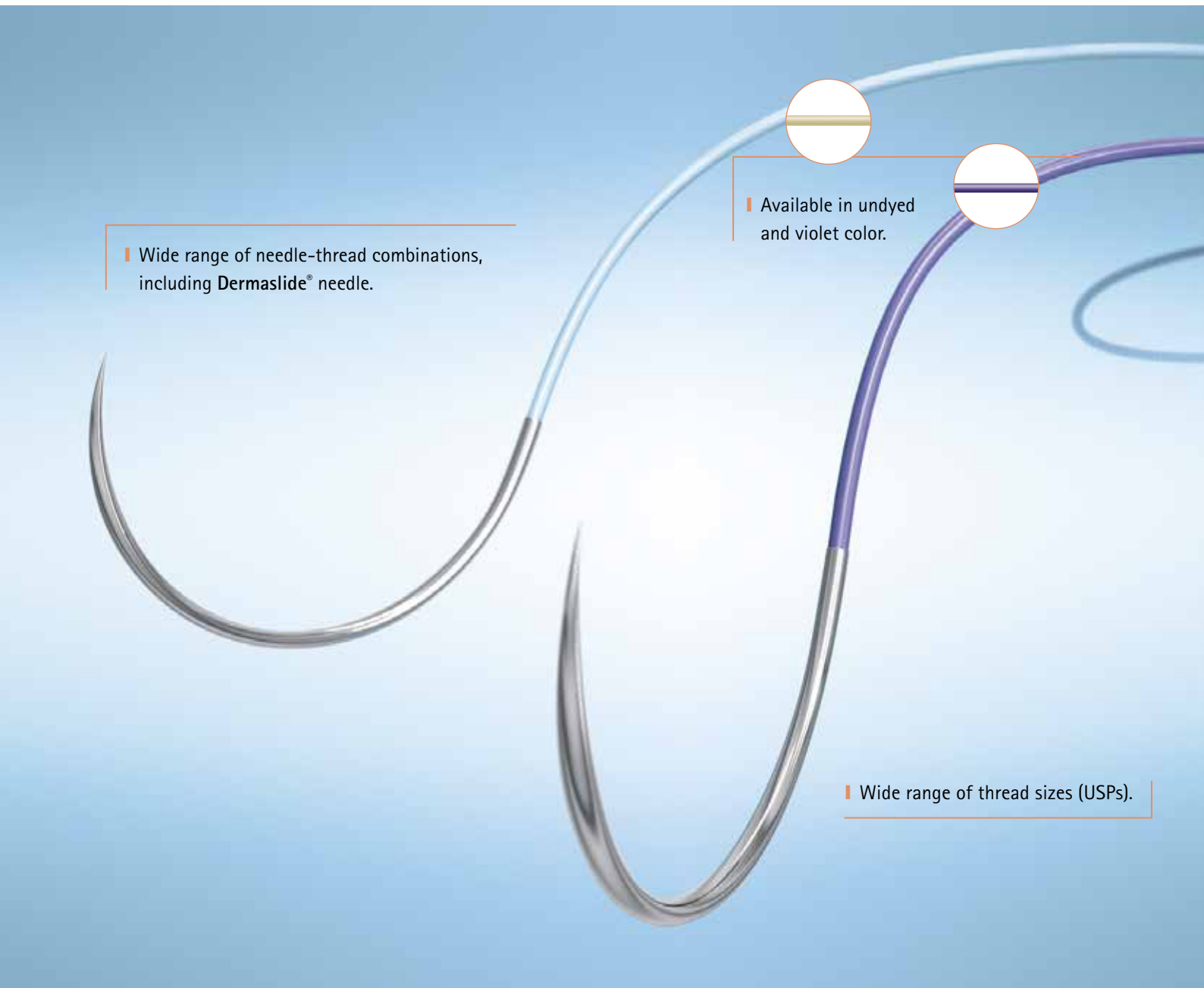
The main advantages of using monofilament sutures:

- The monofilament sutures appear to have the lowest infection promoting effects compared to polyfilament sutures (1-4).
- Compared with braided multifilament sutures, absorbable monofilaments exhibit less tissue drag and cause less tearing because of their smooth surfaces (5).



Monosyn[®]

YOUR CHOICE FOR SOFT TISSUE APPROXIMATION



Wide range of needle-thread combinations, including Dermaslide[®] needle.

Available in undyed and violet color.

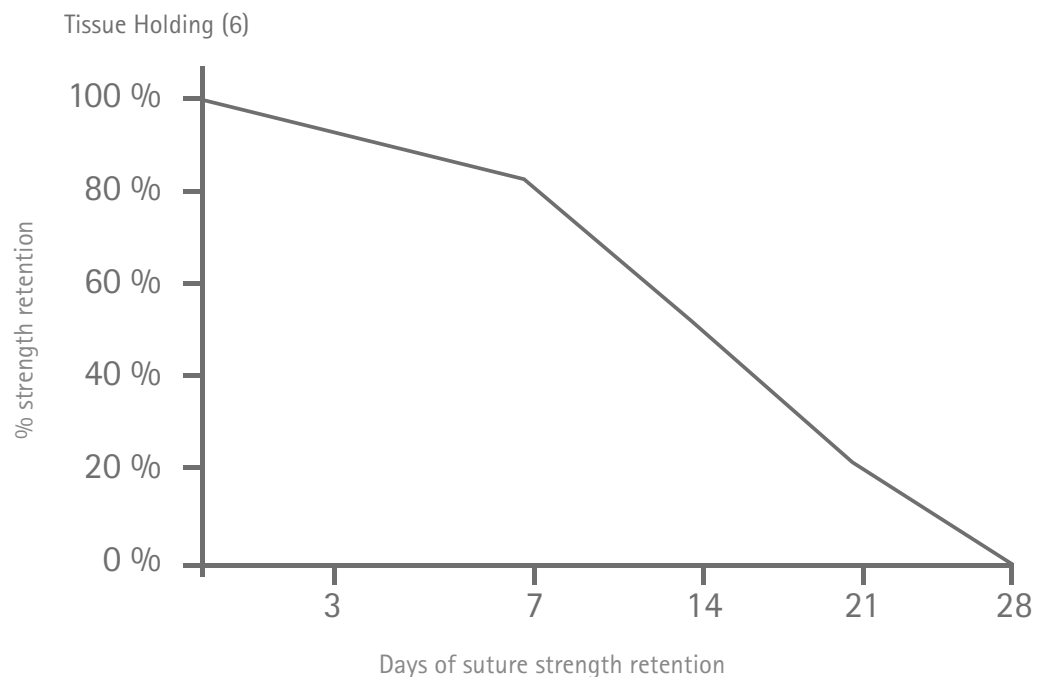
Wide range of thread sizes (USPs).

FEATURES OF Monosyn®

- High knot pull tensile strength (KPTS). Monosyn® KPTS is 4-6% higher than the market standard (6).
- Implantation studies have shown that Monosyn® retains approximately 50% of the original knot pull tensile strength after 13 to 14 days (6).

The mass degradation of Monosyn® is essentially completed in 60 to 90 days (7).

ABSORPTION PROFILE OF Monosyn®



INDICATIONS (8)

Monosyn® is indicated for use in general soft tissue approximation and/or ligation in:

- General surgery
- Gynecology
- Oral surgery
- Skin closure
- Also used in children and neonates (9-14)

Monosyn®

NEW Dermaslide® NEEDLE: A DEFINITIVE CHOICE FOR SKIN CLOSURE

FROM OUR FIRST REVERSE CUTTING TO THE OUTSTANDING Dermaslide® NEEDLE:

- Made from 300 series steel.



THE REVERSE CUTTING NEEDLE TIP WITH PROGRESSIVE MICRO-POINT ENHANCES IT SO THAT EVERY PUNCTURE IS MADE WITH:

- Excellent tissue penetration (15).
- High bending performance (15).



- The DS needle was our first launch indicated for skin closure.
- It is our traditional reverse cutting needle with extensive experience in the market: more than 15 million units delivered around the world annually.

THE FLATTENED AND ENGRAVED BODY OF THE Dermaslide® NEEDLE PROVIDES:

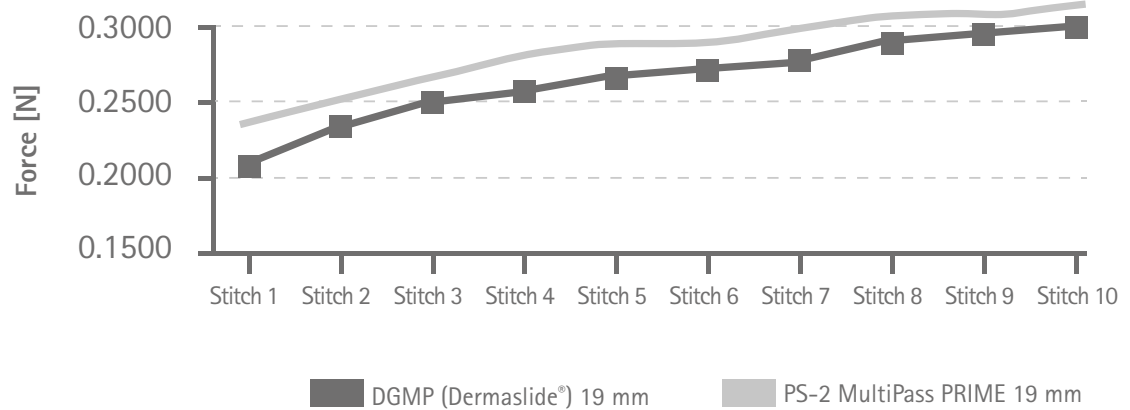
- Improvement in the stability of the needle in the grasp of the needle holder (16).





- Improved tissue penetration (15).
- High bending performance (15).
- Sharper after multiple stitches compared to classical cutting needle (15).*
- Designed to improve the stability of the needle in the grasp of the needle holder (16).

Dermaslide® PENETRATION PERFORMANCE (17)



* B. Braun classical cutting needles compared with B. Braun Dermaslide® needles

A surgical suture should include a needle-thread combination that offers high performance to surgeon for the desired indication.

B. Braun has gone a step further by offering an innovative plastic surgery needle for skin closure to combine with Monosyn®.



Monosyn®

YOUR CHOICE FOR SOFT TISSUE APPROXIMATION



RacePack®

Monosyn® IN RacePack® PACKAGING

RacePack® is a single step packaging system which provides:

- Low memory effect (18).
- Easy product identification.



PRODUCT OVERVIEW







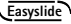




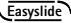

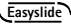



Monosyn®

Structure	Monofilament
Color	Undyed, Violet
Chemical composition	Glyconate, a polymer made of: 72 % Glycolide 14 % Trimethylene carbonate 14 % ε-caprolactone
Sizes	USP 2 (metric 5) -7/0 (metric 0.5)
Knot tensile strength retention	70 % after 7 days 50 % after 14 days 20 % after 21 days
Mass absorption	Degradation by hidrolysis within 60-90 days
Sterilisation	Ethylene oxide



Monosyn®

NEEDLE-THREAD COMBINATIONS



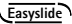



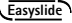

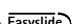




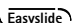
Needle length mm	Thread length cm colour	USP (metric)								
		7/0 (0.5)	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	
1/2 circle round bodied needle										
 HR 10 	70 violet		C0022123							
	70 violet			C2022008						
	45 4 x 45 70 violet	C0022132	C0022019 C0022007	C0022008	C0022009					
	70 undyed		C0023007	C0023008	C0023009					
 HR 13b 	70 violet		C0022301							
 2xHR 13 	70 violet		C0022801	C0022800						
	70 violet			C2022003	C2022004	C2022005				
	70 90 120 violet		C0022002 C0022238 C0022257	C0022003 C0022810 C0022259	C0022004 C0022811 C0022262					
	70 undyed			C0023003	C0023004	C0023005	C0023006			
 HR 17b 	70 violet			C0022616	C0022688	C0022689				
 HR 17 	4 x 45 8 x 45 violet	take off			C0022501 M0022938	C0022502 M0022939				
 2xHR 17 	70 violet			C0022803	C0022804	C0022805				
 2xHR 17b 	70 violet				C0022626					

C Box of 36 units

B Box of 24 units




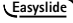



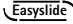


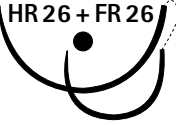
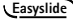

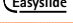

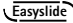
G Box of 12 units

M Box of 6 units


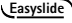

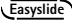

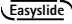

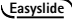

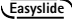
Needle length mm	Thread length cm colour	USP (metric)								
		7/0 (0.5)	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	
1/2 circle round bodied needle										
	 70 violet				C2022014	C2022015	C2022016			
	30					C0022017	C0022018			
	45		C0022412	C0022413		C0022415				
	70			C0022013	C0022014	C0022015	C0022016			
	90				C0022240	C0022246				
	120 violet				C0022263	C0022519	C0022520			
 70 undyed				C0023014	C0023015	C0023016				
	 70 violet				C2022659					
	8 x 45					M0022148				
	70 violet				C0022659	C0022660	C0022661			
	 4 x 45 8 x 45 4 x 70 violet				C0022510 M0022695 C0022061	C0022511 M0022696	C0022512			
	 8 x 45 4 x 70 violet				M0022627 C0022064					
	 70 90 violet				C0022814 C0022914	C0022815 C0022915				
	90 violet					C0023915				
	90 undyed									
	 70 90 violet				C0022632 C0022638	C0022639				
		70 90 violet				C2022024	C2022025 C2022425	C2022026 C2022426	C2022027	
		30					C0022022	C0022023		
70					C0022024	C0022025	C0022026	C0022027	C0022028	
90				C0022423	C0022422	C0022425	C0022426	C0022250		
120 violet				C0022524	C0022424	C0022525	C0022526			
 70 undyed				C0023024	C0023025	C0023026				

Monosyn®

NEEDLE-THREAD COMBINATIONS


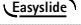

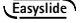

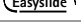
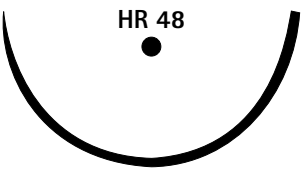
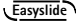
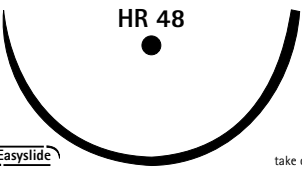
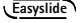

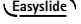
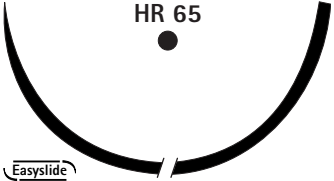
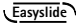
Needle length mm	Thread length cm colour	USP (metric)								
		7/0 (0.5)	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	1 (4)
1/2 circle round bodied needle										
 HR 26b 	70 violet					C2022357				
	8 x 45 70 violet					C0022643	M0022153 C0022357	C0022358		
 HR 26 	4 x 45				C0022521	C0022522	C0022523			
	8 x 45				M0022260	M0022261				
	4 x 70 violet				C0022571	C0022572				
 HR 26b 	8 x 45					M0022645				
	8 x 70 violet					M0022646				
 2xHR 26 	90 violet					C2022825				
	20							C0022296		
	70				C0022823	C0022835	C0022836			
	90 violet				C0022824	C0022825	C0022826			
 2xHR 26b 	70					C0022649	C0022654			
	90 violet					C0022650				
 HR 26 + FR 26 	90 violet						C0022218			
 HR 26s 	70 violet						C2022086			
	70 violet				C0022084	C0022085	C0022086	C0022087	C0022088	
Needle length mm	Thread length cm colour	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)	
 HR 26s 	70									
	90 violet				C0022485	C0022486	C0022487	C0022488	C0022089	

C Box of 36 units	B Box of 24 units	G Box of 12 units	M Box of 6 units
-------------------	-------------------	-------------------	------------------

Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
1/2 circle round bodied needle									
 HR 30 	70 violet				C2022035	C2022036	C2022037		
	70 90 violet			C0022034	C0022035	C0022036 C0022496	C0022037 C0022497	C0022038	C0022039
	70 undyed						C0023037		
	4 x 45 violet				C0022585				
 2xHR 30 	70 90 violet				C0022435 C0022535	C0022436 C0022536			
 HR 30s 	70 violet							C0022042	
 HR 37 	70 90 violet				C2022445	C2022046	C2022047	C2022048	
	70 90 violet			C0022044	C0022045 C0022445	C0022046 C0022446	C0022047 C0022447	C0022048 C0022448	C0022049
	70 undyed						C0023045		
	4 x 45 violet					C0022542	B0022543		
 HR 37s 	90 violet					C2022456	C2022457	C2022458	
70 90 violet				C0022055	C0022056 C0022456	C0022057 C0022457	C0022058 C0022458		
90 120 undyed				C0023930	C0023456	C0023081		B0023470	

Monosyn®

NEEDLE-THREAD COMBINATIONS

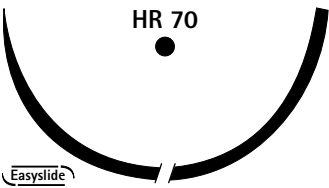
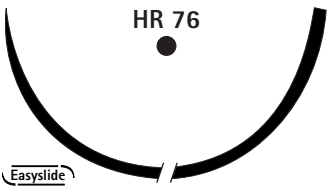






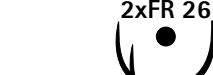
Needle length mm	Thread length cm colour	USP (metric)								
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)	
1/2 circle round bodied needle										
 HR 37s 	4 x 70 undyed					C0023546				
	70 90 violet						C2022067 C2022467	C2022068 C2022468		
 HR 40s 	70 90 violet					C0022066 C0022466	C0022067 C0022467	C0022068 C0022468	C0022069 B0022469	
	70 90 violet									
 HR 40s 	4 x 45 violet						B0022548			
	90 violet							C2022078		
 HR 48 	90 120 140 150 violet					C0022076	C0022077	C0022078 B0022578 B0022290	B0022079	
	120 undyed						B0023479	B0023478		
 HR 48 	4 x 45 violet							B0022554		
 HR 60 	90 violet							B0022490		
 HR 65 	90 violet						B0022492			


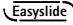

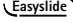

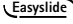

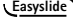

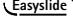

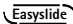

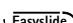

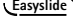

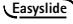

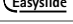

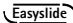


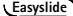

C Box of 36 units

B Box of 24 units













G Box of 12 units

M Box of 6 units

Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
1/2 circle round bodied needle									
 HR 70 Easyslide	70 violet								B0022760
 HR 76 Easyslide	90 150 violet								B0022493 B0022494
5/8 circle round bodied needle									
 FR 13 Easyslide	90 violet		C0022606	C0022607					
 FR 17 Easyslide	90 violet		C0022610	C0022611					
 FR 22 Easyslide	90 violet		C0022614	C0022615					
 FR 26 Easyslide	70 violet					C2022620	C2022621		
	70 90 violet			C0022618	C0022619	C0022620 C0022625	C0022621		
 2xFR 26 Easyslide	70 90 violet			C0022734	C0022735 C0022727	C0022736 C0022740			
 FR 36 Easyslide	70 violet				C0022635	C0022636	C0022637		
	70 undyed						C0023618		
 2xFR 36 Easyslide	90 violet					C0022729	C0022730	C0022731	

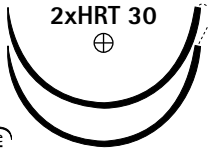

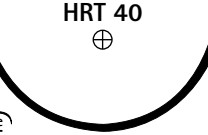
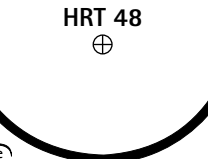
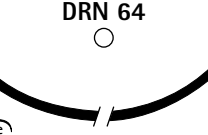
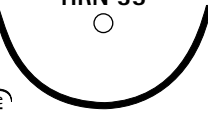
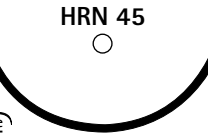
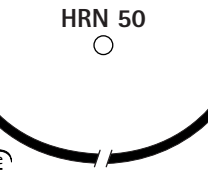
Needle length mm	Thread length cm colour	USP (metric)								
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)	
5/8 circle round bodied needle										
 FR 36 + FR 26 	90 violet						C0022151			
 FR 40 	70 90 violet						C0022629	C0022630 C0022670		
Straight round bodied needle										
 GR 19 	30 violet			C0022410	C0022406					
 GR 65 	90 violet					C0022679				B0022302
 2xGR 65 	70 violet						C0022575			
Ski Needle round bodied needle										
 SKR 19 	120 violet					C0022714				
 SKR 26 	30 120 violet					C0022715 C0022717	C0022716 C0022718			
3/8 circle round bodied needle with short cutting point										
 DRC 10 	70 violet	C0022382								
 DRC 12 	70 violet	C0022392	C0022403							
 2xDRC 12 	70 violet	C0022832	C0022845							
 DRC 22  	70 violet			C2022354	C2022355					
 DRC 26  	70 violet						C2022366			

C Box of 36 units	B Box of 24 units	G Box of 12 units	M Box of 6 units
-------------------	-------------------	-------------------	------------------

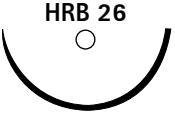
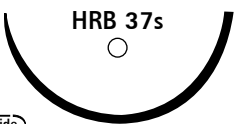
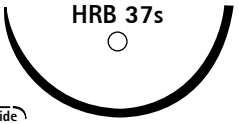
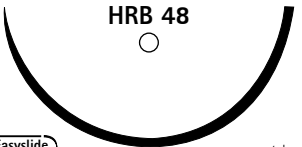






Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
1/2 circle round bodied needle with short cutting point									
 HRC 26 Easyslide	70 violet				C2022305				
3/8 circle round bodied needle with trocar point									
 DRT 18 Easyslide	70 violet		C0022343	C0022344					
 DRT 22 Easyslide	70 violet			C0022354	C0022355				
 DRT 26 Easyslide	70 violet				C0022365	C0022366			
 DRT 32 Easyslide	70 violet					C0022376	C0022377	C0022378	
1/2 circle round bodied needle with trocar point									
 HRT 13 Easyslide	70 violet	C0022310	C0022311						
 HRT 17 Easyslide	70 violet	C0022322	C0022323	C0022324	C0022325	C0022326			
 2xHRT 17 Easyslide	70 violet			C0022724	C0022725				
 2xHRT 22 Easyslide	70 90 violet			C0022330 C0022430	C0022331 C0022431				
 HRT 26 Easyslide	70 90 120 violet			C0022304	C0022305	C0022306 C0022255 C0022300	C0022307 C0022507	C0022308 C0022508	
 2xHRT 26 Easyslide	70 violet			C0022404	C0022405				
 HRT 30 Easyslide	70 violet				C0022335	C0022336	C0022337	C0022338	

Monosyn®

NEEDLE-THREAD COMBINATIONS








Needle length mm	Thread length cm colour	USP (metric)								
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)	
1/2 circle round bodied needle with trocar point										
 2xHRT 30 ⊕ Easyslide	70 violet				C0022440	C0022441				
 HRT 37 ⊕ Easyslide	70 90 violet				C0022315	C0022316 C0022516	C0022317 C0022517	C0022318 C0022518		
 HRT 40 ⊕ Easyslide	90 violet						C0022527	C0022528		
 HRT 48 ⊕ Easyslide	90 150 violet						C0022332	C0022333 B0022334		
3/8 circle round bodied needle with blunt point										
 DRN 64 ○ Easyslide	90 violet								B0022400	
1/2 circle round bodied needle with blunt point										
 HRN 35 ○ Easyslide	70 violet					C0022283				
 HRN 45 ○ Easyslide	90 violet								C0022438	
 HRN 50 ○ Easyslide	90 violet									C0022453

C Box of 36 units	B Box of 24 units	G Box of 12 units	M Box of 6 units
-------------------	-------------------	-------------------	------------------


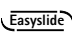

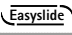

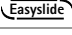

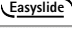

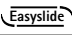

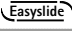

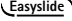
Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
1/2 circle round bodied needle with protection point									
 HRB 26 Easyslide take off	8 x 45 violet				M0022871				
 HRB 37s Easyslide	90 violet						C0022707	C0022708	
 HRB 37s Easyslide take off	8 x 45 violet					M0022287	M0022890	M0022891	
 HRB 48 Easyslide take off	8 x 45 violet						M0022880	M0022881	
3/8 circle reverse cutting needle									
 DS 12 Easyslide	 70 undyed	C2023412	C2023413	C2023414					
	45	C0022102	C0022103	C0022104					
	70 violet	C0022109	C0022210						
	45 70 undyed	C0023102 C0023412	C0023103 C0023413	C0023414					
 DS 16 Easyslide	 45 violet		C2022419						
	 45			C2023303					
	 70 undyed		C2023423	C2023424	C2023425				
	45 70 violet		C0022419	C0022420 C0022220	C0022421				
	45 70 undyed	C0023562	C0023563 C0023423	C0023303 C0023424	C0023425				

Monosyn®

NEEDLE-THREAD COMBINATIONS


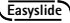

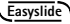
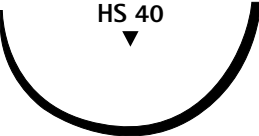
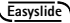
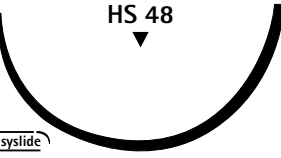
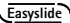

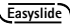
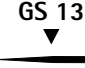
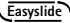
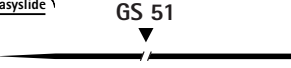
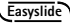

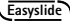
Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
3/8 circle reverse cutting needle									
 DS 19	 45 70 violet			C2022504 C2022204	C2022205				
	 45 70 undyed		C2023403	C2023204 C2023404	C2023205 C2023405				
	45 70 90 120 violet		C0022503	C0022504 C0022204 C0022904 C0022081	C0022505 C0022205 C0022905 C0022264	C0022239			
	45 70 undyed		C0023503 C0023403	C0023204 C0023404	C0023205 C0023405	C0023506 C0023406			
	 70 violet				C2022215	C2022216	C2022217		
	 70 undyed			C2023214	C2023215	C2023216			
	45 70 90 120 violet			C0022214	C0022115 C0022215 C0022225 C0022267	C0022216 C0022226 C0022286	C0022217 C0022251 C0022289		
	45 70 90 undyed			C0023114 C0023214	C0023115 C0023215 C0023210	C0023116 C0023216	C0023181	C0023182	
	 70 violet					C2022231	C2022232	C2022233	
	70 90 120 violet					C0022231 C0022252 C0022291	C0022232 C0022349 C0022293	C0022233	
45 70 undyed			C0023207	C0023435	C0023436	C0023437			
 DS 39	70 undyed					C0023191			

C Box of 36 units	B Box of 24 units	G Box of 12 units	M Box of 6 units
-------------------	-------------------	-------------------	------------------





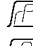



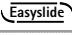


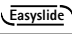


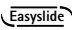
Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
1/2 circle reverse cutting needle									
 HS 18 	70			C0022234					
	90 120 violet			C0022254 C0022295					
	70 undyed		C0023263	C0023264					
 2xHS 18 	30 undyed			C0023674					
 HS 23 	70 undyed				C2023225				
	70 undyed			C0023224	C0023225	C0023226			
 HS 26 	70 90 violet				C2022236 C2022072	C2022237 C2022299			
	70 90 120 violet				C0022236 C0022072 C0022080	C0022237 C0022299 C0022074			
	70 undyed				C0023239	C0023240			
	70 violet					C2022242	C2022243		
 HS 26s 	70 90 120 violet				C0022241	C0022242 C0022053 C0022075	C0022243 C0022052 C0022082		
	70 undyed							C0023230	
	70 undyed								
	70 undyed								
 HS 30 	70 undyed					C0023250	C0023251		
 HS 37 	70 undyed				C0023235	C0023236	C0023237	C0023238	

Monosyn®

NEEDLE-THREAD COMBINATIONS



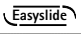


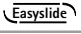


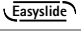


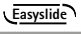
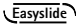
Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
1/2 circle reverse cutting needle									
 HS 37s	70 violet						C2022248	C2022249	
	70					C0022247	C0022248	C0022249	
	90					C0022461	C0022477	C0022478	
	120 violet					C0022297	C0022108	C0022136	
	70 undyed					C0023243	C0023244		
 HS 37ss	70 violet					C0022256			
									
 HS 40	70								C0022309
	90								B0022269
	120 violet								B0022298
	90 undyed						C0023247	C0023248	
									
 HS 48	90 violet								B0022279
	90 undyed						C0023259	C0023258	
									
5/8 circle reverse cutting needle									
 FS 25	70 undyed					C0023220			
									
Straight reverse cutting needle									
 GS 13	45 undyed			C0023131					
									
 GS 51	70 undyed			C0023279	C0023280	C0023281			
									
 GS 60	70 undyed				C2023285				
	70 violet				C0022062				
	70 undyed			C0023284	C0023285	C0023286			
									

C Box of 36 units	B Box of 24 units	G Box of 12 units	M Box of 6 units
-------------------	-------------------	-------------------	------------------

Needle length mm	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
3/8 circle Trapezoidal micro engraved with precision point									
DGMP 11 	45 undyed	C0023709	C0023705						
DGMP 13 	45 undyed	C0023710	C0023706	C0023702					
DGMP 16 	45 70 undyed		C0023707	C0023703	C0023711				
DGMP 19 	 45 70 undyed			C2023704	C2023712 C2023701				
	45 70 undyed		C0023708	C0023704 C0023012	C0023701				
DGMP 24 	 70 undyed				C2023717				
	70 undyed				C0023717				
Needle length mm	Thread length cm colour	USP (metric)							
		8/0 (0.4)	7/0 (0.5)	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)
3/8 circle reverse cutting needle with precision point									
DSMP 7 	45 violet		C0022647	C0022648					
	 45 undyed		C0023647						
DSMP 11 	 45 undyed			C2023642	C2023643				
	45 violet		C0022644	C0022601	C0022602				
	 45 undyed		C0023641	C0023642	C0023643				
DSMP 13 	 45 undyed				C2023603	C2023604			
	45 violet		C0022608	C0022432	C0022433	C0022434			
	 45 undyed		C0023601	C0023602	C0023603	C0023604	C0023605		

Monosyn®

NEEDLE-THREAD COMBINATIONS

Needle length mm	Thread length cm colour	USP (metric)							
		8/0 (0.4)	7/0 (0.5)	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)
3/8 circle reverse cutting needle with precision point									
 DSMP 16	 45 70 undyed				C2023613	C2023614 C2023163			
	45 violet				C0022563				
	 45 70 45 undyed				C0023613 C0023162 C0023612	C0023614 C0023163			
 DSMP 19	 45 70 undyed					C2023624 C2023627	C2023625 C2023628		
	45 70 violet					C0022444 C0022141			
	 45 70 undyed				C0023623	C0023624 C0023627	C0023625 C0023628		
 DSMP 24	 45 70 undyed					C2023634 C2023632	C2023633 C2023635	C2023636	
	70 violet						C0022685		
	 45 70 undyed					C0023634 C0023632	C0023633 C0023635	C0023637 C0023636	
 DSMP 24s	70 undyed						C0023638		
1/2 circle reverse cutting needle with precision point									
 HSMP 9	45 undyed			C0023651		C0023608			
 HSMP 15	45 undyed					C0023615			
									

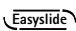





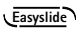





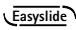

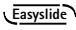



C Box of 36 units

B Box of 24 units

G Box of 12 units

M Box of 6 units

	Thread length cm colour	USP (metric)							
		6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	2 (5)
Pre-cut sutures									
3 threads of 45 cm	3 x 45 violet		C0020003	C0020004	C0020005	C0020006	C0020007	C0020008	
6 threads of 45 cm	6 x 45 violet			C0020014	C0020015	C0020016	B0020017	B0020018	
5 threads of 70 cm	5 x 70 violet			C0020024	C0020025	C0020026	C0020027	B0020028	
1 thread of 140 cm	1 x 140 violet				C0020035	C0020036	C0020037	C0020038	
6 threads of 45 cm	6 x 45 undyed					C0023811			
Ligature reels									
1 thread of 250 cm	1 x 250 violet					G0020046	G0020047	G0020048	
1 thread of 250 cm	1 x 250 undyed					G0021246	G0021247		

Needle length mm	Thread length cm colour	USP (metric)								
		7/0 (0.5)	6/0 (0.7)	5/0 (1)	4/0 (1.5)	3/0 (2)	2/0 (3)	0 (3.5)	1 (4)	
1/2 circle round bodied needle										
 HR 10 	70 violet		C0022123							
	70 violet			C2022008						
	45 4 x 45 70 violet		C0022132	C0022019 C0022007	C0022008	C0022009				
	70 undyed			C0023007	C0023008	C0023009				
 HR 13b 	70 violet		C0022301							
 2xHR 13 	70 violet		C0022801	C0022800						
	70 violet			C2022003	C2022004	C2022005				
	70 90 120 violet			C0022002 C0022238 C0022257	C0022003 C0022810 C0022259	C0022004 C0022811 C0022262				
	70 undyed				C0023003	C0023004	C0023005	C0023006		
 HR 17b 	70 violet			C0022616	C0022688	C0022689				
 HR 17 	4 x 45 8 x 45 violet	take off				C0022501 M0022938	C0022502 M0022939			
 2xHR 17 	70 violet			C0022803	C0022804	C0022805				
 2xHR 17b 	70 violet					C0022626				

C Box of 36 units	B Box of 24 units	G Box of 12 units	M Box of 6 units
-------------------	-------------------	-------------------	------------------

- (1) Mangram AJ, Horan TC, Pearson ML, Silver LC, Jarvis WR. Guideline for Prevention of Surgical Site Infection, 1999. Centers for Disease Control and Prevention (CDC) Hospital Infection Control Practices Advisory Committee. *Am J Infect Control.* 1999;27(2):97-132.
- (2) Choi HJ, Chae HD. Comparison of E. Coli infiltration between new synthetic absorbable sutures. *J Korean Surg Soc.* 2009;77(1):1-6.
- (3) Buresch A, Van Arsdale A, Ferzli M, Bernstein P, Garry D, Ngai I. Comparison of subcuticular suture type in post-cesarean wound complications: A randomized controlled trial *American Journal of Obstetrics and Gynecology.* 2017;216:1 Supplement 1 (S25).
- (4) Won H-S, Lee S-W, Kim Y-M, Kim A. Clinical usefulness and safety of the anti-bacterial coated multi lamant suture (Vicryl Plus®) and mono lamant suture (Monosyn®) in hysterectomy *BJOG: An International Journal of Obstetrics and Gynaecology.* 2012;119 SUPPL. 1 (44)
- (5) Im JN, Kim JK, Kim HK, Lee KY, Park WH. Characteristics of novel monofilament sutures prepared by conjugate spinning. *J Biomed Mater Res B Appl Biomater.* 2007;83(2):499-504.
- (6) RDR/02/FF1526/JLT-C.
- (7) Data on file: In vivo test Ra 572.
- (8) IFU Monosyn®.
- (9) Van der Linden YTK, Boersmal D, Van Poll D, Lips DJ, Prins HA. Single-Port Laparoscopic Appendectomy in Children: Single Center Experience in 50 Patients. *Acta Chirurgica.* 2015;115:118-22.
- (10) Oh C, Lee S, Lee SK, Seo JM et al. Laparoscopic duodenoduodenostomy with parallel anastomosis for duodenal atresia. *Surg Endosc.* 2016;31:2406.
- (11) Koo EJ, Jung E, Choi SO. Laparoscopic Choledochal Cyst Excision and Hepaticojejunostomy: A Case Series. *J Minim Invasive Surg.* 2017;20(2):58-62.
- (12) Lee S, Lee SK, Seo JM. Thoracoscopic repair of esophageal atresia with tracheoesophageal fistula: Overcoming the learning curve. *Journal of Pediatric Surgery.* 2014;49(11):1570-2.
- (13) Oh C, Lee S, Lee SK, Seo JM. Difference of Postoperative Stool Frequency in Hirschsprung Disease According to Anastomosis Level in a Single-Stage, Laparoscopy-Assisted Transanal Endorectal Pull-Through Procedure. *Medicine (Baltimore).* 2016;95(14):e3092.
- (14) Salmerón-González E, García-Vilariño E, Gutiérrez-Ontavilla P, Miró-Rubio I, López-Blanco E. Evaluation and comparison of two different cartilage-sparing techniques in 356 otoplasties performed in children. *Annals of Pediatric Surgery.* 2018;14(3):143-5.
- (15) B. Braun Test Report 113_139/A Benchmark Cosmetic Needles: Ethicon-Covidien-SetR.
- (16) B. Braun R&D Testing Services Report (RDR-22-13110-FJB-CMY).
- (17) Data on file (2013), determined by Penetration measurement protocol SXMAD/05/A 20/03/06.
- (18) RDR/02/FF1511/JLT-A.

B. Braun Surgical, S.A. | Carretera de Terrassa, 121 | 08191 Rubí | Spain
Tel. +34 935 86 62 00 | +34 935 88 10 96 | www.bbraun.es

Information according to Regulation (EU) 2017/745 and Council Directive 93/42/EEC respectively:



Manufacturer

B. Braun Surgical, S.A. | Carretera de Terrassa, 121 | 08191 Rubí | Spain

The product trademark "Dermaslide", "Monomax", "MonoPlus", "Monosyn" and "Racepack" are registered trademarks of B. Braun Melsungen AG or its affiliates.

CE 0123