

# Medibox® – How to use

## LID ASSEMBLY



Place the lid on the medical waste container and press firmly on the opposite corners of the lid until a „click“ confirms the lock.

Check that the lid is properly locked on all sides!


## DISPOSAL



Open the lid by applying slight pressure on the tab – the lid flap jumps up slightly.



Push up the lid flap with one hand, until it locks into a standing position.

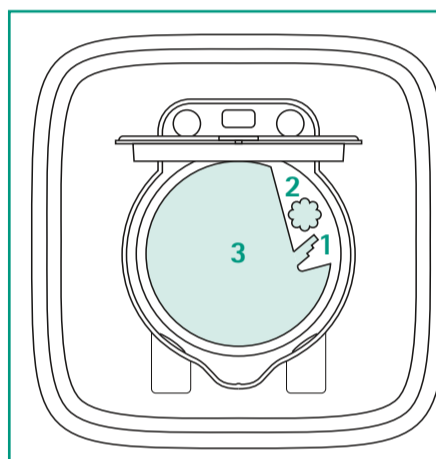

### DISPOSAL OF

1. luer and luer-lock needles
2. pen needles
3. various medical sharps



1 Disconnect luer and luer lock connections between syringe and cannula.



2 Unscrew a pen needle from a pen.

3 Dispose large medical sharps.

## LID CLOSURE



**TEMPORARY CLOSURE:**  
Release the lid flap from the standing position and press it lightly onto the housing.



**PERMANENT CLOSURE:**  
Close the Medibox® after reaching the maximum filling volume (see marking line on lid) by simultaneously pressing on the marked pressure points.



Dispose the permanently closed Medibox® in accordance with your local regulations.

In order to view the Medibox® handling video directly on your mobile phone and to obtain product information, simply scan the QR code with the camera of your smartphone and a suitable reader application and get to the website.

