

Disinfection Instructions For CARESITE® Luer Access Device

Three simple steps to flush or administer medication:

1. Wash Hands



- Use antibacterial soap.
- Wear gloves for IV procedure.

2. Disinfect Valve Top



- Vigorously swab with alcohol pad; best practice recommends 15 seconds.
- Allow alcohol to dry.

3. Attach and Disconnect



- **IMPORTANT:**
Do not clamp line until syringe is disconnected.
- Remove any residual fluid upon disconnect with alcohol pad.

Always Remember:

- Swab valve top vigorously with an alcohol pad for 15 seconds before each infusion access.
- Flush before and after meds.
- Flush before and after blood draws.
- Disinfect valve top after each blood drawing process is complete.
- Upon completion of IV infusion follow institutional protocol for IV care and valve change.
- Always refer to package instructions for complete directions for use.

For questions or information, contact B. Braun
Clinical/Technical Support at **1-800-854-6851**

B | BRAUN
SHARING EXPERTISE

www.bbraunusa.com