

Priming of Caresite® Double Extension Sets

Method with Prefilled Flush Syringe



1 Gather all the required equipment such as Caresite® extension set, Omniflush®, Softa® Cloth and tray for disposal.



2 Use aseptic technique. Disinfect both Caresite® valves per national and/or institutional protocol.



3 Activate Omniflush® and connect it to one Caresite® valve.



4 Prime the first lumen of the extension set.



5 Connect Omniflush® to the second Caresite® valve.



6 Prime the second lumen of the extension set.



7 The priming process is completed. Remove protective cap of the Caresite® extension set.



8 Connect the Caresite® extension set to the IV catheter.



9 Start infusion therapy.



B | BRAUN
SHARING EXPERTISE