

EZCOVER[®] Quick Guide

Generic Ultrasound Probe Cover Set



1 Open pouch and unwrap set




2 Locate middle fold of the sleeve by looking for the "Hand here" sticker



3 Insert hand into the cover



4 a Apply a small amount of sterile gel on either sleeve (a) or on the probe (b)

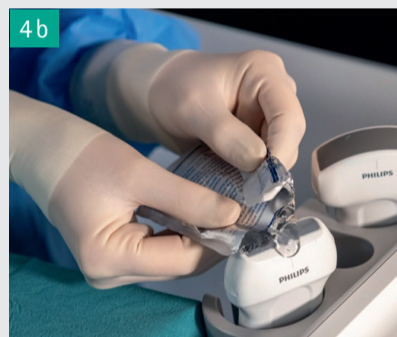

5 a Pick up the Ultrasound transducer (a*) or have it be handed to you (b)
*Note: "probe here" visible for STOP (Check) variants only



6 Pull the cover over the transducer and cable ("pull here" sticker)



7 Fixation with rubber bands



8 Perform time out and remove peel off sticker (STOP (Check) variants only)



9 a For clear ultrasound images make sure that there are no air bubbles under the probe cover